



USER MANUAL

PHACON TEMPORAL BONE ASSISTANT

INSTRUCTIONS

INTENDED USE

The PHACON TemporalBone Assistant is intended to be used by healthcare professionals for education and training purposes only. The assistant allows surgical residents and other healthcare professionals to use actual surgical instruments on three dimensional (3-D) anatomical models.

If you are not sure whether parts are damaged or faulty, or if you have any questions regarding the use, please contact PHACON Support (Phone: +49 (0)341 47 83 97 32 or support@phacon.de) in immediately.

USER SAFETY



WARNINGS:

- Only trained and experienced healthcare professionals should use this equipment. Before using any system component, or any component compatible with this assistant, read and understand the instructions. Become familiar with the assistant components prior to use.
- Upon initial receipt and before each use, operate the equipment and inspect each component for damage. DO NOT use any component if damage is apparent. See the care / cleaning instructions section.

NOTE: If you need more information:

Phone: +49 (0)341 47 83 97 32

Mail: support@phacon.de



OTHER GENERAL WARNINGS

Always use your surgical instruments according to the intended use of the manufacturer.

Cases and packaging

WARNINGS:

The suitcase and the packaging foils are not toys and not suitable for children. There is a risk of suffocation!


The packaging material is not suitable for eating.

Packaging waste is not to be disposed of via normal household waste. Please think about your environment.

Please do not use any liquid food or drink during use.



DISCOVER THE INTRODUCTION
IN OUR PHACON TEMPORAL BONE ASSISTANT

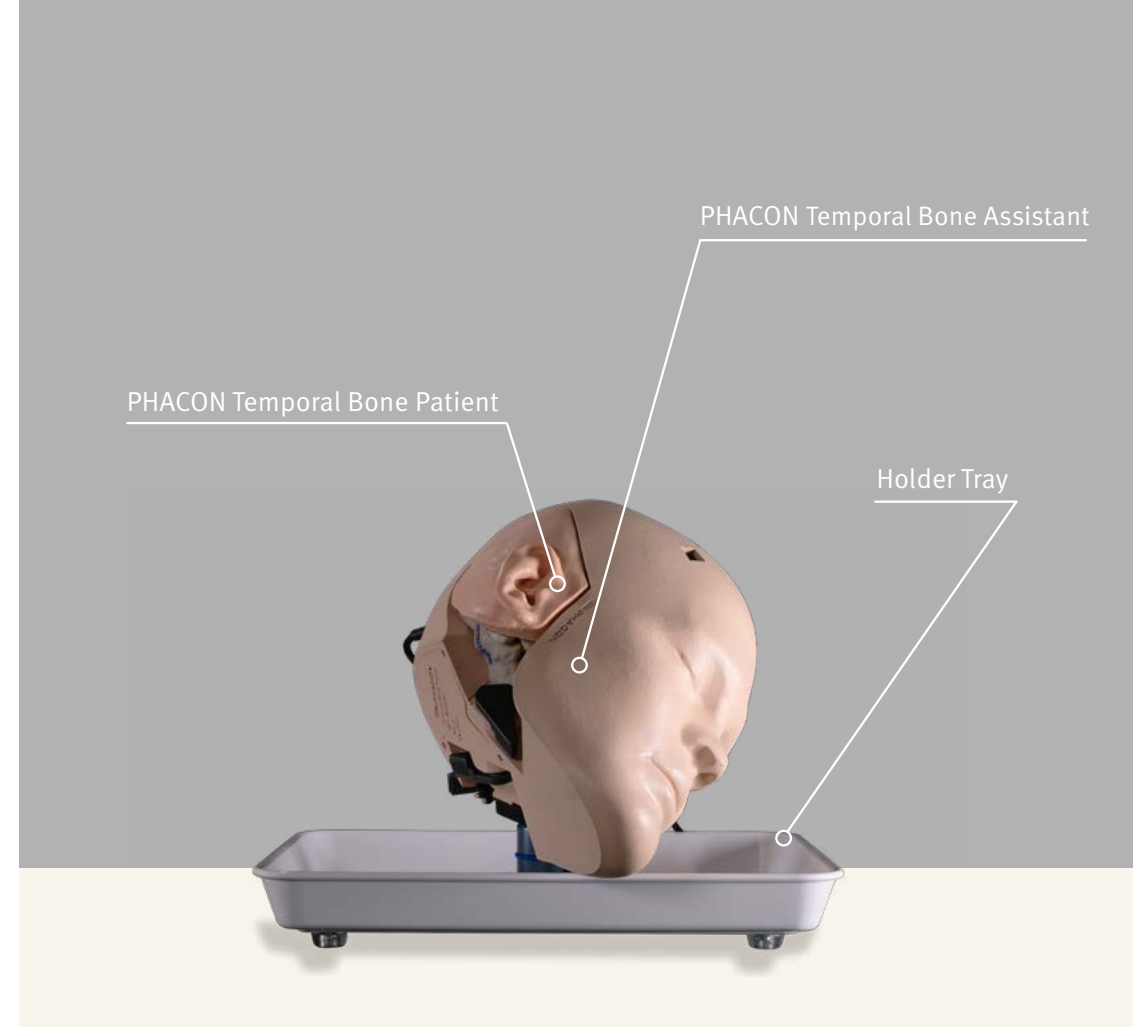

PACKING INSTRUCTIONS

PHACON TEMPORAL BONE ASSISTANT - TRANSPORT CASE

1. First insert the manual into the recess. Then insert the holder tray on the same position.



3. You can place the PHACON Temporal Bone Patient in the holder tray like in the picture.



4. Cover holder tray and the patient with the second layer of foam, containing a recess for the tracker box.







5. Insert the two head adapter into the recess like on the picture.



6. Place the head in the free space on the top of the holder tray. Now you can close the case lid top gently.



TEMPORAL BONE ASSISTANT COMPONENTS

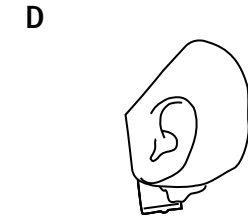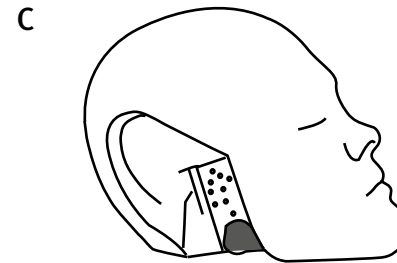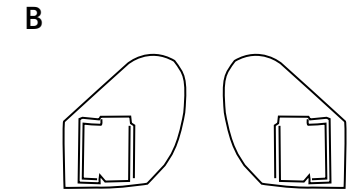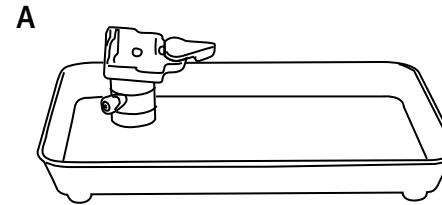

A Holder Tray

B Mounting Adapter

C PHACON Temporal Bone Assistant

D PHACON Temporal Bone Patient „Schmidt“ [TF-by]

INSTRUCTIONS

Image 1


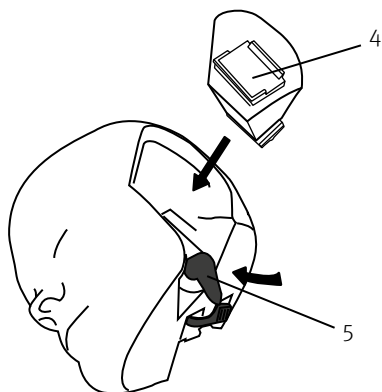


Image 2

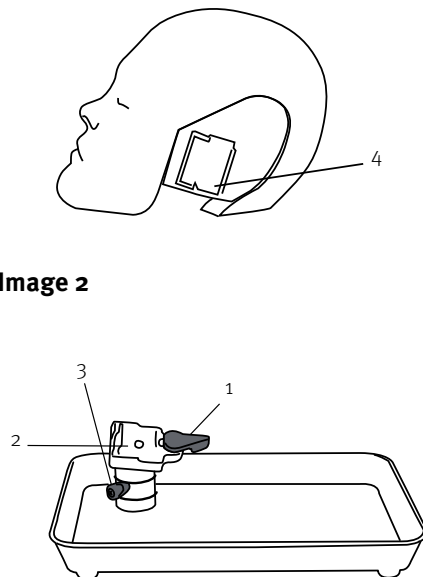

LEGENDE

1. Locking lever

Release to attach the head.

2. Mounting plate

To fix the head. The head plate is strapped into the mounting plate.

3. Swivel switch

For adjusting the head position. Tighten to lock the head.

4. Head plate

The mounting plate is located on the mounting adapter. This will be lockt in the head.

5. Locking lever of Assistant

Release to attach the patient and the mounting adapter.

6. Patient


Slide the patientplate in the head.

1. First fix the mounting adapter in the PHACON Temporal Bone Assistant

Image 1 Turn the locking lever on the Assistant of the holder tray (5) to the left. Slide in the mountingplate (4) of the mounting adapter. Close the locking lever of the mounting plate to the right again. Open the locking lever to the left. You can also use this principle for other side. First fix the adapter in the holding shell and then fix the patient in the Head.

INSTRUCTIONS

Image 3

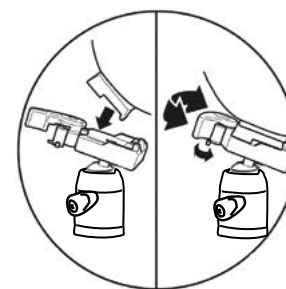

2. Second fix the Holder Tray

Image 2 . Second you fix the mounting plate, for this the ball head is brought into the desired position. Now tighten the swivel switch counterclockwise.



TIP:

Make sure to fasten it properly to prevent it from falling over.

3. Fasten the head to the Holder Tray


Image 3 Turn the locking lever on the mounting plate base of the holder tray to the left. Slide in the plate of the head. Close the locking lever of the mounting plate to the right again. Open the locking lever to the left.

Image 4

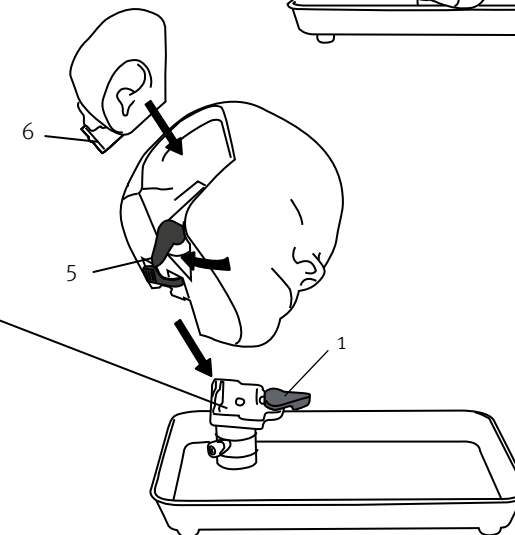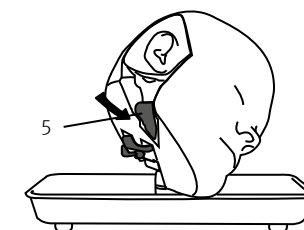

Fix the (right) patient in the PHACON Temporal Bone Assistant

Image 4 The same principle is used to attach the patient and the Mounting bracket to the Temporal Bone Assistant. Turn the locking lever on the Assistant of the holder tray (5) to the left. Slide in the patientplate (6) of the Temporal Bone patient. Close the locking lever of the mounting plate to the right again. Open the locking lever to the left. You can also use this principle for the left side. First fix the adapter in the holding shell and then fix the patient in the head.

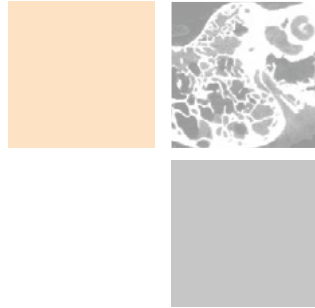

THE PATIENTS FOR YOUR SURGICAL DEMONSTRATION AND EDUCATION

EUROPE

PHACON GmbH

Druckereistrasse 11
04159 Leipzig
Germany

Phone: +49 (0)3 41 47 83 97 32
Fax: +49 (0)3 41 47 83 91 74
Mail: info@phacon.de

www.phacon.de

USA

PHACON Inc.

4470 Chamblee-Dunwoody Rd. /// Suite 463
Dunwoody /// GA 30338 /// USA

Phone: 1 833 2 PHACON
2 742266
Fax: 470 243 0494
Mail: info@phacon.us

www.phacon.us